WIKKO MIKRO

Mikrochirurgisches Instrumentarium für Parodontologie und Endodontie
Microsurgical Instruments for Periodontics and Endodontics

Mikrochirurgisches Grundinstrumentarium nach PD Dr. H. Wachtel Microsurgical basic set acc. to PD Dr. H. Wachtel

Erfahrung

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Experience

For decades, microsurgical instruments have been developed and continuously improved by AESCULAP for microsurgery.

Material and temper

AESCULAP micro instruments are made of high-grade stainless steel on the basis of many years of experience.

Thus a decisively higher temper is obtained for scissors compared to other materials, for example titanium.

Additional diamond powder coating improves the grip and extends the working life of AESCULAP needle holders and tissue forceps.

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DIADUST needle holders

- optimized gripping of needles and sutures
- long life time, since the degree of the temper of the diamond powder coating is nearly identical to the temper of the needles
- for suture material 6/0 - 9/0

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*Micro needle holder, curved,
diamond powder coated, for
suture material 6/0 - 8/0*

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*Micro needle holder, straight,
diamond powder coated, for
suture material 6/0 - 8/0*

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*Micro needle holder, straight,
diamond powder coated, for
suture material 8/0 - 9/0*

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DIADUST forceps

- non-traumatic and yet firm gripping of sutures and tissues.

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*Micro dissecting forceps,
diamond powder coated
plateau, corresponding with
micro needle holder FD 258 R*

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*Micro dissecting forceps,
diamond powder coated
plateau, corresponding with
micro needle holder FD 268 R*

new Neue Instrumente mit ca. 30%
verkleinerten Arbeitsenden.
New instruments with working
tips reduced by approx. 30%

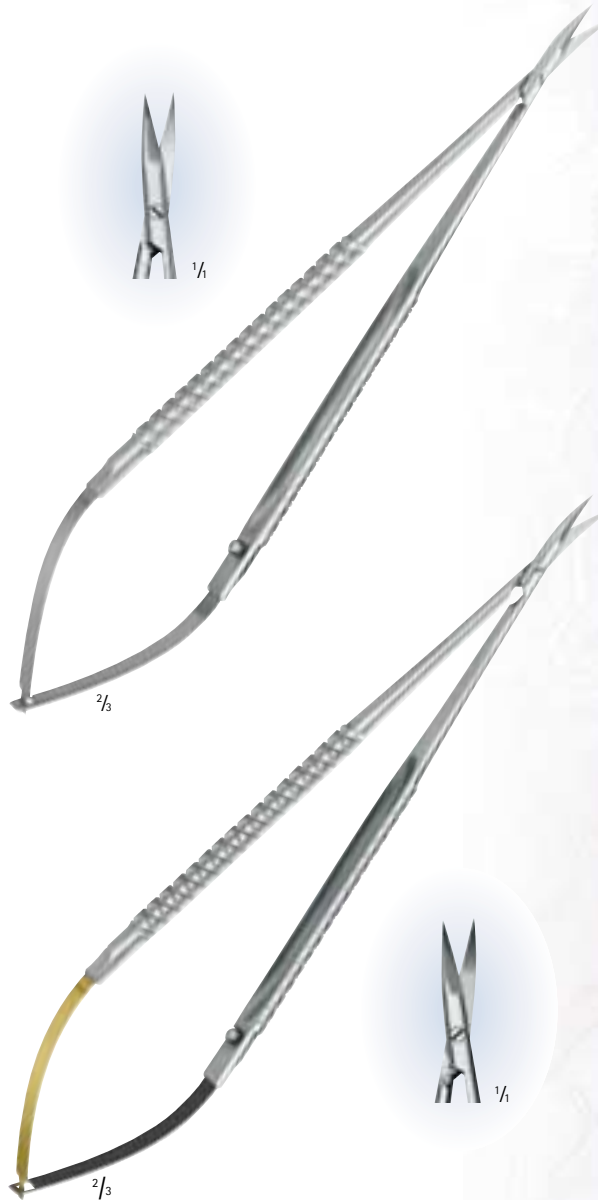
Mikrochirurgisches Grundinstrumentarium nach PD Dr. H. Wachtel Microsurgical basic set acc. to PD Dr. H. Wachtel

FD 251 R 180 mm
Mikro-Federschere, gebogen,
spitz/spitz

*Micro scissors (spring type),
curved, sharp/sharp*

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*Micro ligature scissors, curved,
sharp/sharp, one blade serrated,
for suture material 6/0 - 8/0*

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Scissors

Screw connection providing:

- smooth and reliable function
- longer life time by the possibility of regrounding

Special hint

Since suture material is rather tough, the cutting capacity of scissors is quickly reduced which is noticeable as soon as tissues are cut afterwards.

Our solution

To cut sutures with special ligature scissors. The ligature scissors FD 252 R have additionally rounded edges to avoid sutures to get jammed.

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*Micro tissue forceps, curved,
diamond powder coated
plateau, corresponding with micro
needle holder FD 258 R*

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*Micro tissue forceps, curved,
diamond powder coated
plateau, corresponding
with micro
needle holder
FD 268 R*

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*Micro tissue forceps, straight,
diamond powder coated
plateau, corresponding with
micro needle holder FD 258 R*

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*Micro tissue forceps, straight,
diamond powder coated
plateau, corresponding with micro
needle holder FD 268 R*

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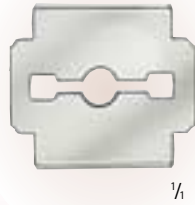
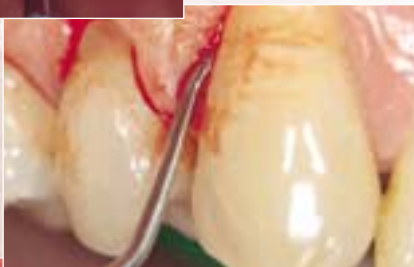
*Papilla-incision
 with the micro
 scalpel.*

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
*Atraumatic dissection of
 the flap by means of the
 papilla elevator without
 any damage to the
 papillae.*

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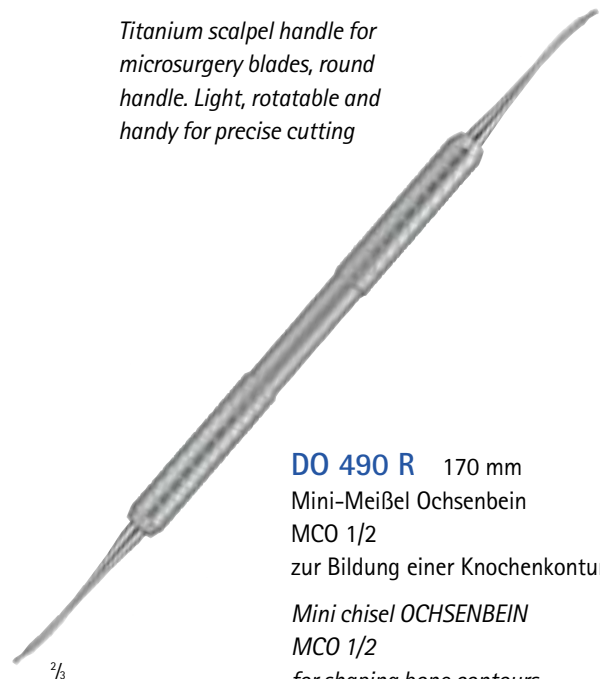
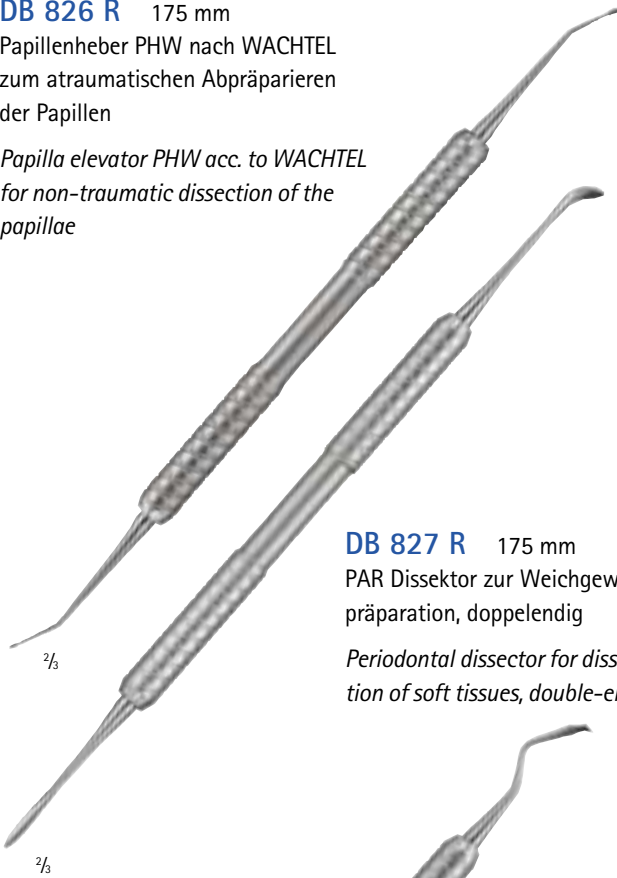
*Papilla elevator PHW acc. to WACHTEL
for non-traumatic dissection of the
papillae*

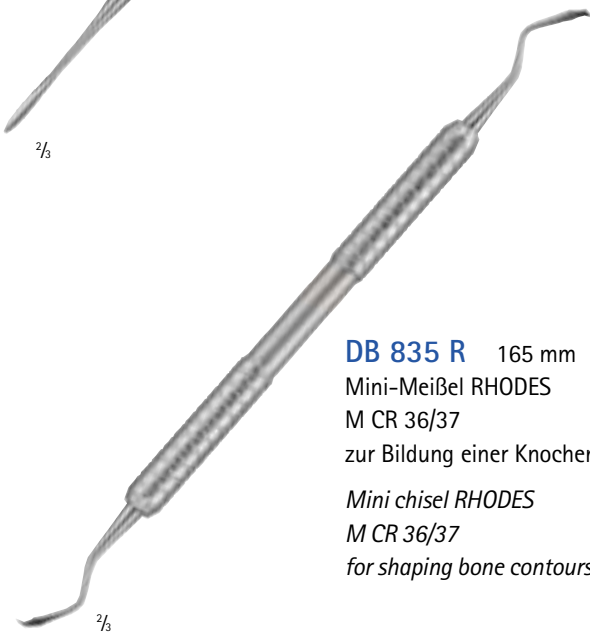DB 827 R 175 mm
PAR Dissektor zur Weichgewebe-
präparation, doppelendig
*Periodontal dissector for dissec-
tion of soft tissues, double-ended*

DB 835 R 165 mm
Mini-Meißel RHODES
M CR 36/37
zur Bildung einer Knochenkontur
*Mini chisel RHODES
M CR 36/37
for shaping bone contours*

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*Mini chisel KRAMER NEVINS
M CKN 1/2
for dissection of soft
and hard tissues*

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*State after mechani-
cal treatment of the
root surface.*

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*Complete flap closure
after a healing period
of 5 days, the inter-
dental papilla being
maintained.*

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Endodontic microsurgical set acc. to Dr. P. Velvart

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Elevation of the flap starting from the relieving incision with the disk-shaped elevator.

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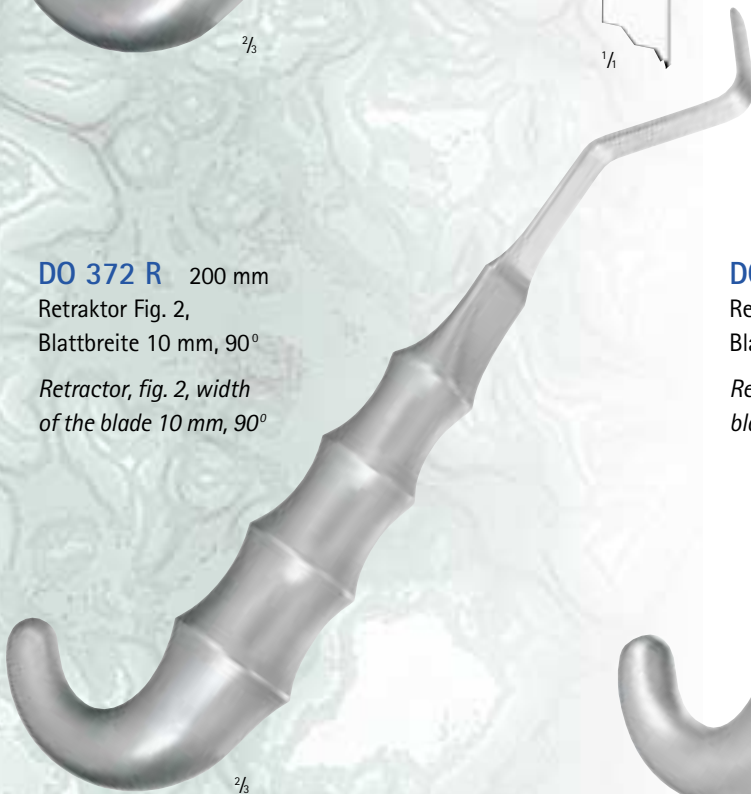
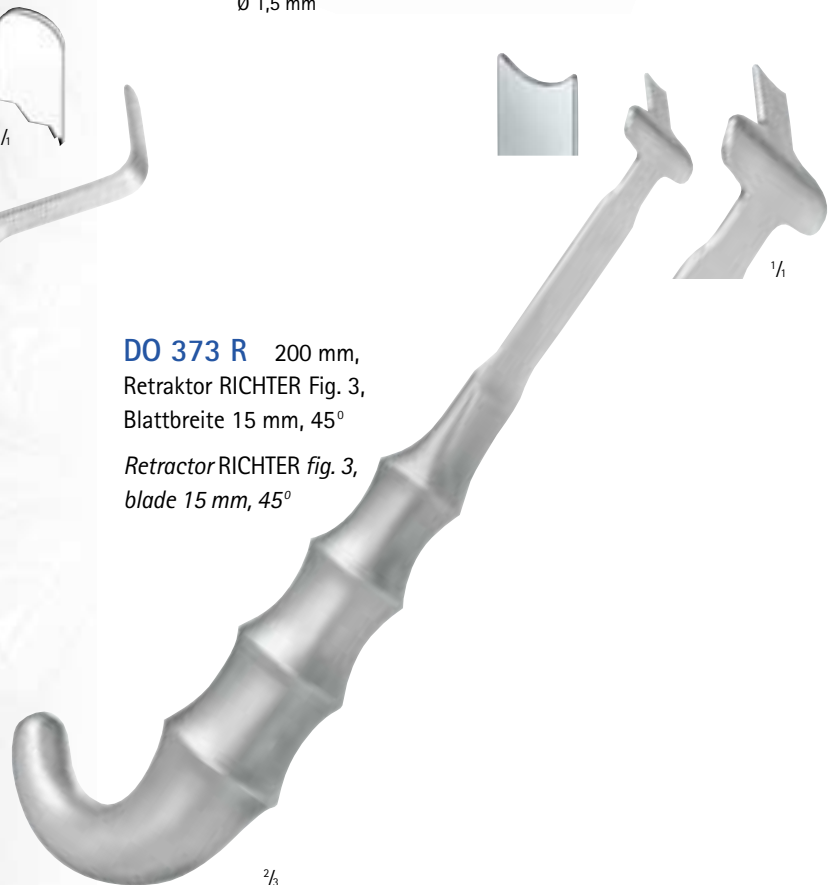
*Disk-shaped elevator
and dissector*

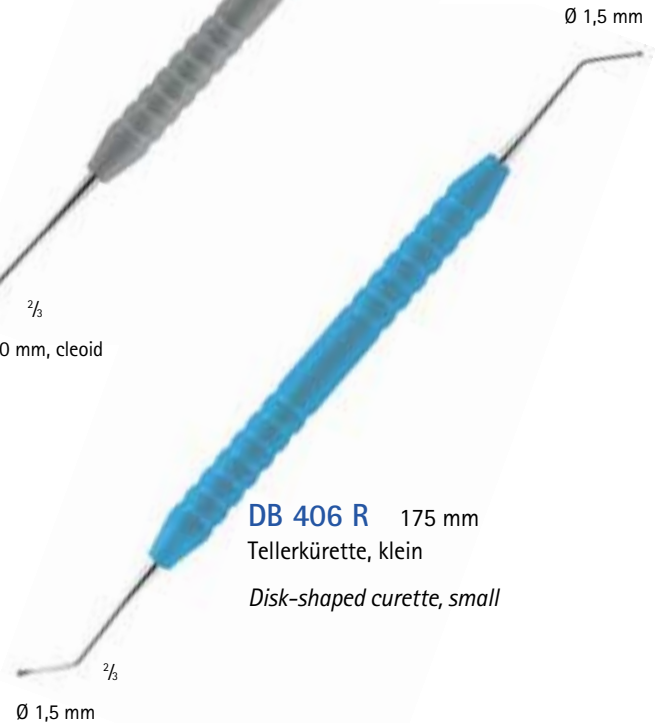
Ø 3,0 mm, cleoid

Ø 1,5 mm


DB 406 R 175 mm

Tellerkürette, klein

Disk-shaped curette, small


Ø 1,5 mm

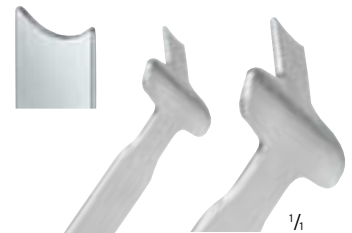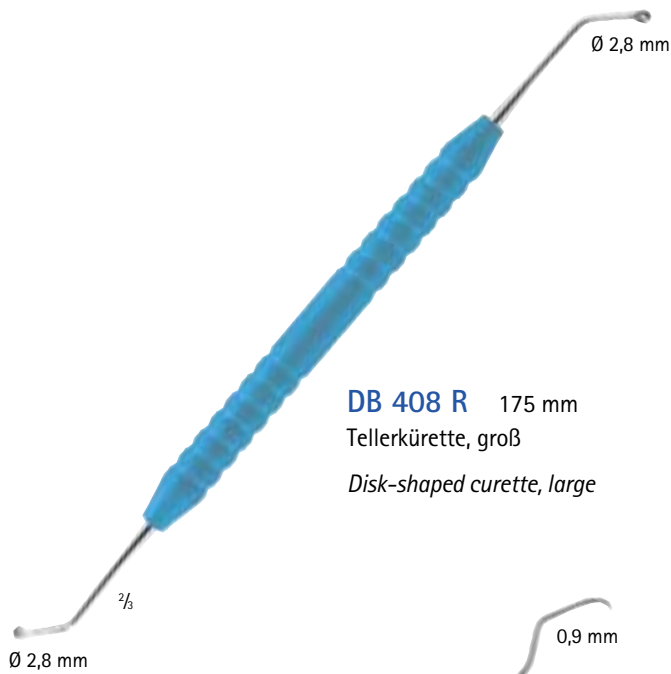Mikrochirurgisches Instrumentarium für die Endodontie nach Dr. P. Velvart Endodontic microsurgical set acc. to Dr. P. Velvart


DB 408 R 175 mm
Tellerkürette, groß
Disk-shaped curette, large

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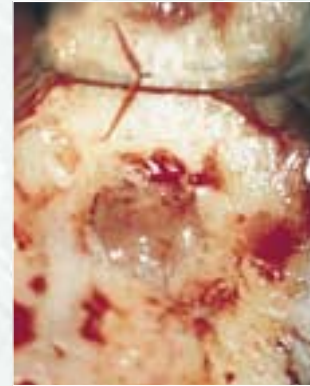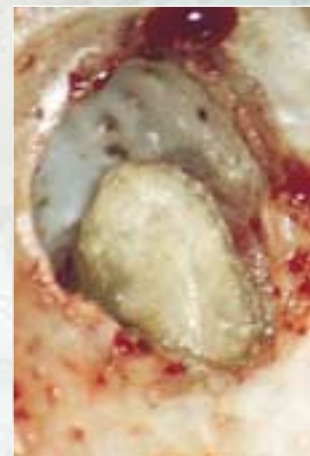
*Retraction of the flap using the retractors.
The apical lesion is visible in the center of the picture.*

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
The apical lesion removed using the following: disk-shaped curettes, the spoon-shaped curette or the angular scaler.

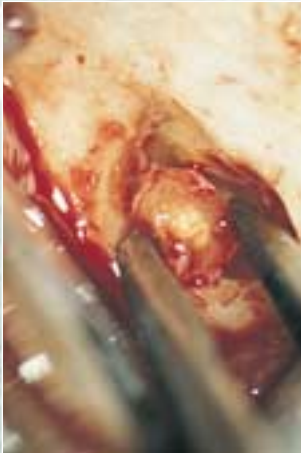Mikrochirurgisches Instrumentarium für die Endodontie nach Dr. P. Velvart
Endodontic microsurgical set acc. to Dr. P. Velvart

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
Removal of the resected apex with the surgical micro forceps.

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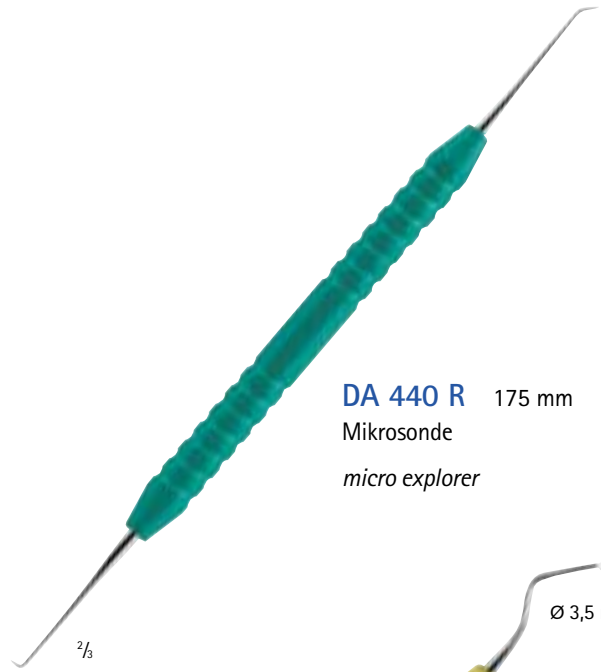
Probing of the canals with the micro explorer.

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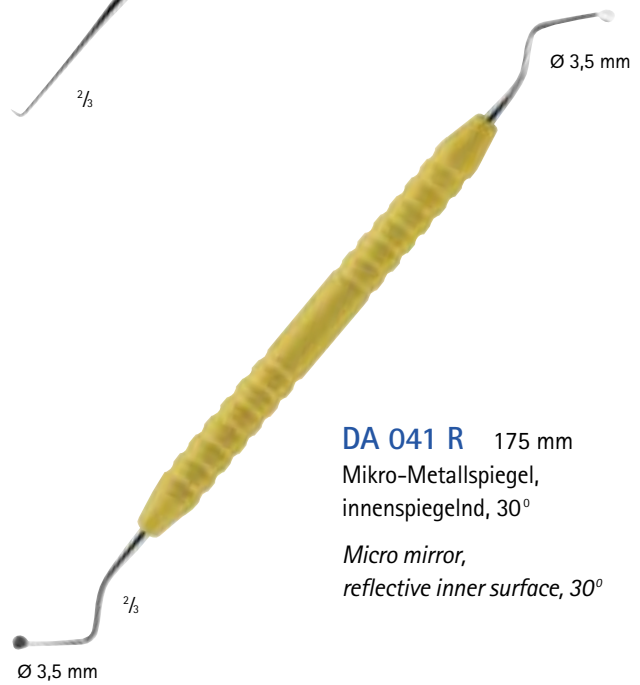*Handle for interchangeable micro
mirrors DA 043 R - DA 046 R
and for microsurgery blades*

Mikrochirurgisches Instrumentarium für die Endodontie nach Dr. P. Velvart Endodontic microsurgical set acc. to Dr. P. Velvart

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Micro mirror, round, 3.5 mm diam., 35° bevel, interchangeable, PZ = package of 10 pieces

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Micro mirror, round, 4.7 mm diam., 35° bevel, interchangeable, PZ = package of 10 pieces

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Micro mirror, 2 mm wide, 45° bevel, interchangeable, PZ = package of 10 pieces

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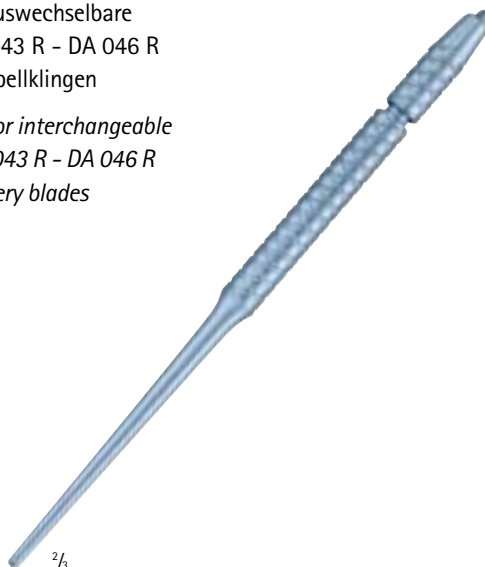
Micro mirror, 3 mm wide, 45° bevel, interchangeable, PZ = package of 10 pieces

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Use of the oval micro mirror. Fracture line barely detectable in low magnification (12x).

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Large magnification (25x) of the above illustration. The fracture line is visible in the center of the illustration.

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Use of the round micro mirror. View of prepared retrograde cavity with the post in the root canal.

Mikrochirurgisches Instrumentarium für die Endodontie nach Dr. P. Velvart Endodontic microsurgical set acc. to Dr. P. Velvart

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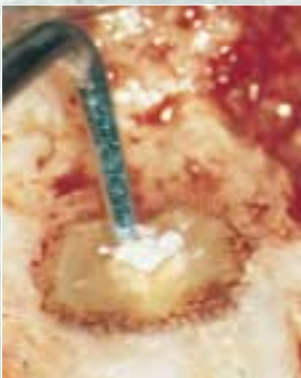
Oval retrograde cavity, seen after preparation of the isthmus between the root canals. The oval shape plugger is used to condense the retrograde filling material.

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
Application of the retrograde filling material (Super EBA) into the cavity with the micro spatula.

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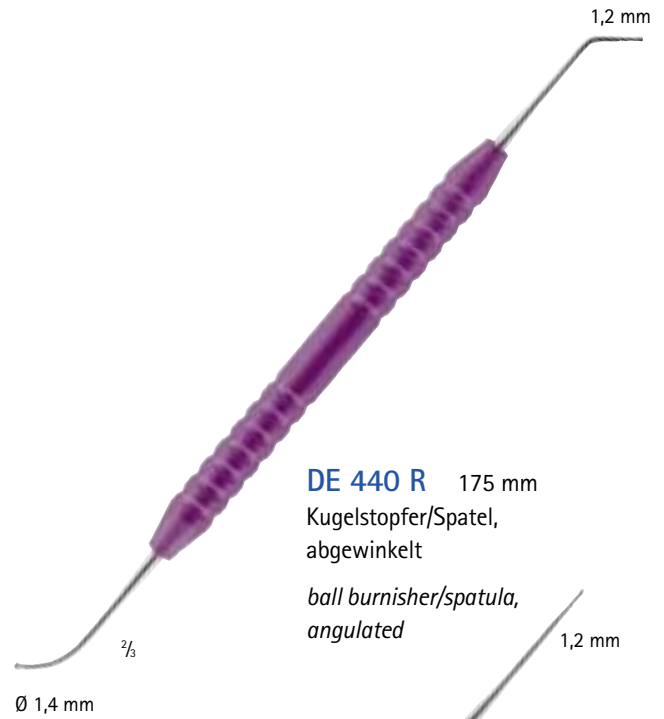Condensation of the filling material with the round plugger (in round canals).

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AESCULAP AG & CO. KG